



# Instructions for Use

## Intermittent Coude Male Catheter

**Before you begin, gather the supplies you need to insert the catheter. You will need:**


- TruCath™ Intermittent Coude Catheter
- HR® Lubricating Jelly 3g OneShot
- Something to cleanse the area: soap, water and a wash cloth, or unscented wipes
- A container to collect the urine. (If you empty the urine right into the toilet, you won't need the container.)
- Hand towel



### How to Use

*If able, try to pass urine and empty your bladder as much as you can before using an intermittent catheter.*

1. Wash hands thoroughly with soap and water and dry them.
2. Position yourself comfortably. If sitting on the toilet, spread your legs apart. If lying down, knees can be straight or bent.
3. Wash the urethral area from front to back with soap and water or an unscented wipe.
4. Apply lubrication to catheter with HR® Lubricating Jelly 3g OneShot packet.
  - Tear off top of the lubrication packet
  - Apply lubricant to tip and along the length of the catheter (first several inches of catheter)
5. Remove the catheter from the package.
6. Angle the coude tip in the 12 o'clock position and insert into the urethra. Gently slide the catheter forward and use the notched funnel as a guide until you reach the bladder — you will feel some resistance when you reach the bladder sphincter. Continue to insert until urine starts to flow.
7. Hold in place until urine stops flowing. Ensure the funnel end is directed into a toilet or collection receptacle. When drainage is complete, slowly remove the catheter.
8. Dispose of the catheter in a waste bin - do not flush it down the toilet. Wash your hands with soap and water.

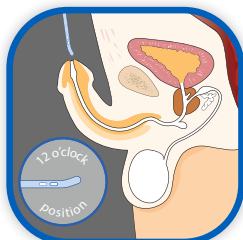

*Consult your healthcare provider if you notice any unusual odor and/or cloudy, hazy urine.*

*The TruCath™ Intermittent Coude Catheter is intended for one-time use.*