

AquaFlate™

Pre-Filled Sterile Water Syringe 10 mL

Instructions for Use

REF

250-400 AquaFlate Pre-Filled Sterile Water Syringe 10 mL

DESCRIPTION

The AquaFlate™ Pre-Filled Sterile Water Syringe is a 10 mL syringe pre-filled with USP purified water and sterilized via gamma irradiation. The syringe consists of a polypropylene barrel and plunger and latex-free rubber for both the plunger gasket and syringe tip cover.

INDICATIONS FOR USE

The AquaFlate Pre-Filled Sterile Water Syringe is intended to be used for inflating a Foley catheter balloon.

CONTRAINDICATIONS

None known.

WARNINGS

- The prefilled syringe containing sterile water is intended to inflate the catheter ONLY and is NOT intended for injection.
- Syringes are for single use only and are not to be reused or re-sterilized.

PRECAUTIONS

- Do not use product if package is damaged or opened.
- If cap is not present, do not use the syringe.
- Syringes should be correctly disposed of after being used once only.
- Read the Instructions for Use and keep them in a safe place.
- Use the product only in accordance with its intended use, see INDICATIONS FOR USE.
- Refer to the Instructions for Use of the Foley catheter manufacturer for specific catheter placement and inflation instructions, including the selection of the appropriate volume of sterile water required to inflate the catheter balloon.

PRE-USE, HANDLING, AND INSPECTION

The product must be thoroughly inspected upon receipt and prior to use to ensure packaging is not damaged or defective.

STEP-BY-STEP INSTRUCTIONS

1. Thoroughly read and follow the Foley catheter manufacturer's instructions for use.
2. When ready to use, remove the syringe cap and follow the Foley catheter manufacturer's instructions for use and inflation.
3. Upon completion of the catheterization, properly discard the syringe.

STORAGE CONDITIONS

- To preserve product performance, store away from direct sunlight.
- Store in a clean, dry place.



HR Pharmaceuticals, Inc.
2600 Eastern Blvd., Suite 201
York, PA 17402 USA
877-302-1110
hrpharma.com



Symbols glossary and additional information available at trucath.com

AquaFlate is a part of the **HR** family of brands.

HR and AquaFlate are trademarks of HR Pharmaceuticals, Inc.
©2023 HR Pharmaceuticals, Inc. All rights reserved.